



Application Instructions

Just dry, apply, wipe!

Thank you for your interest in the DentaSEAL Pro product line! The transparent and odorless material pre-loaded on the wand applicator will easily cover all teeth, a set of dentures (all surfaces), or a variety of removable appliances, etc.

This simple and fast application requires no etching, no bonding agents, no curing and takes less than 10 minutes to apply fully.

NOTES:

- Apply before using fluoride
- Do not apply before teeth have been whitened to the desired level
- Before applying DentaSEAL Pro Sensitivity, be sure to test for and correct any underlying pathologies that might be creating sensitivity, such as a cracked tooth. (see enclosed test sheet)

For all applications, whether as a coating, to address sensitivity, or when applying to removables, follow these directions and you and your patient will enjoy seeing the positive results!

1. Apply to one arch at a time to be able to quickly buff off all residue before it sets.
2. Isolate tooth surfaces to create a drip-free environment and dry teeth with air or cotton.
3. Carefully remove the wand from the tube and use the pre-loaded applicator tip to paint all surfaces you wish to cover.
4. For very hard-to-reach surfaces, swipe material off the applicator tip onto a microbrush.
5. For interproximal application:
 - a) Use a microbrush, expanding floss or dental tape to drag the material from a treated surface and onto interproximal surfaces
 - b) Make back-and-forth/up-down motions with your dental tape or microbrush and pay special attention to interproximal contact points
6. Once each arch is treated, vigorously and thoroughly buff all surfaces with gauze. This creates a slick, thin layer. **The thinner, the better** is how this material works so the more you buff, the better its performance. DentaSEAL Pro creates an instant bond and can't be rubbed off.
7. Repeat the application process on the second arch.
8. If applying for sensitivity, have the patient swish with hot or cold liquid after application. The pain should be resolved. If it is not, repeat and retest. We've had a 100% resolution for sensitivity when no underlying pathologies were present.

For questions please contact:
Steve Turley 801-949-4325
LeeAnn Day 800-603-6003

www.dentasealpro.com

powered by DentaKote NDC#69953-553-01