



www.dentaseal.com

Just dry, apply, and wipe!

This transparent and odorless material pre-loaded on the angled tip of the wand applicator will cover all teeth and restorations, a denture plate (all surfaces), or a variety of removable appliances.

No etching, no bonding agents, and no curing is required. This simple and fast application occurs via a chemical bond (covalence), not a mechanical bond which requires the above actions. Our products generally take only a few minutes to complete a full-mouth or denture application. You may wish to touch the wand tip to your glove to determine if the material is still there; it will leave a small residue mark.

- If using fluoride, apply DentaSEAL products **afterward**
- If whitening teeth, apply DentaSEAL products **after achieving the whitened level desired**
- Correct all underlying pathologies before applying DentaSEAL Protect, i.e. caries, cracked tooth
- Do not use DentaSEAL products in conjunction with other surface coatings or varnish materials

Silicone oil is inert and non-toxic. An allergic reaction is extremely rare. If it occurs, discontinue use and notify the patient's physician.

INSTRUCTIONS FOR USE

1. Isolate tooth surfaces to create a drip-free environment for one arch at a time.
2. Dry teeth with air or cotton (desiccation is not necessary).
3. Carefully remove the wand from the vial, taking care not to touch the angled tip to any surfaces.
4. "Paint" chosen surfaces, one arch at a time; then vigorously buff all surfaces with gauze.
5. Swipe material off the applicator tip with a microbrush for hard-to-reach surfaces.
6. For interproximal areas:
 - a. Use a microbrush, dental tape or expanding floss to drag the material from treated surfaces onto interproximal surfaces by making back-and-forth and up-down motions -- pay special attention to interproximal contact points
7. Once all surfaces are treated, vigorously buff with gauze. This creates a slick, thin layer. The more you buff, the better its performance and protection. **Thinner is better.**
8. Repeat the application process on the second arch.
9. If applying to mitigate sensitivity, have the patient swish with cold or hot water after application. The pain should be resolved; if it's not, repeat the procedure and ensure no underlying pathology exists.

- * Protect and Appliance is for Single Patient Only usage
- * Appliance Multi is for use on new, unworn appliances
- * Dispose of applicator vial and wand after using
- * Store at room temperature

For questions, please contact: Steven Turley @ 833-600-6003